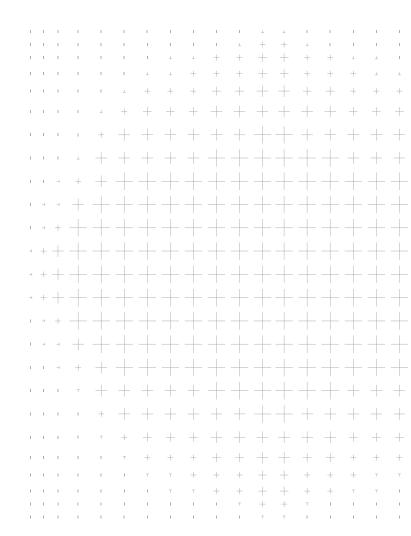
# Advanced Pressure Mapping System

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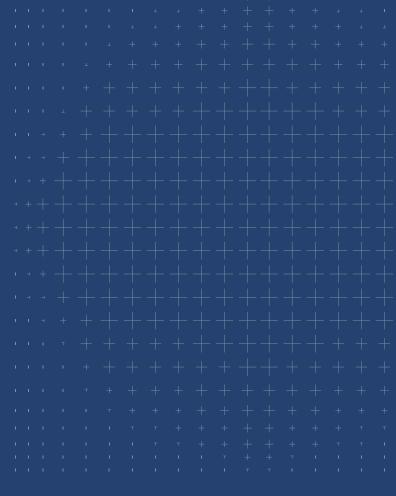
**User Manual** 

#### **Safety Precautions**

Please read all instructions, cautions and warnings displayed in the manual and the product.

#### KongTech Co., Ltd.

Website: http://www.kong-tech.com E-mail: kongtech@kong-tech.com

Tel: +82 2-1522-9928 Fax: +82 70-5143-6695

If you have problems with the product or have any questions, please contact us.

#### Note

- 1. Under the copyright law, this manual can not be copied, reproduced, translated or converted to any other form of media without the written consent of KONGtech Co., Ltd.
- 2. This manual may include print defects and the contents are subject to change without notice.
- 3. KONGtech accepts no liability for any damage, injury or legal responsibility incurred caused by failure to follow the instructions.
- 4. Appearance and specification of the product are subject to change without prior notice to improve performance.

## 

Failure to follow the instructions given with this symbol could possibly result in serious injuries or even death to personnel due to incorrect handling.

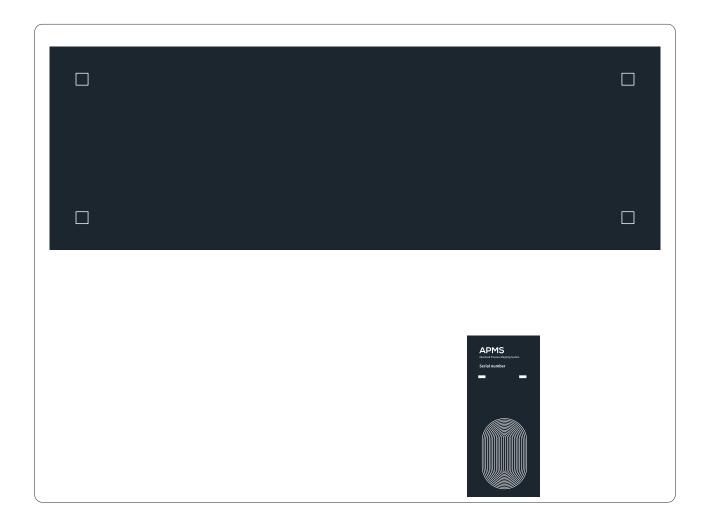
Failure to follow the instructions given with this symbol could possibly result in minor injuries to personnel or damage to the device due to incorrect handling.

# Table of Contents


1. In	1. Introduction	
2. Pa	ckage contents	22
3. Us	Package contents  22  Using APMS  Connecting APMS  Saving measurement data Playing recorded files  Changing measurement data viewing mode  Changing the color of measurement data  Changing the appearance of measurement data  Changing the measurement range of data  Viewing data in a selected range  Continuous viewing  Cother features  Charging  Dimensions  32  Specifications  33  Specifications  34  Specifications  35  Specifications  36  Specifications  36  Specifications  37  Specifications  38  Specifications  Specifications  Continuous of the specifications  Charging  Continuous viewing  Continuous viewing	24
3.1	Connecting APMS	24
3.2	Saving measurement data	29
3.3	Playing recorded files	30
3.4	Changing measurement data viewing mode	31
3.5	Changing the color of measurement data	32
3.6	Changing the appearance of measurement data	34
3.7	Changing the measurement range of data	36
3.8	Viewing data in a selected range	37
3.9	Continuous viewing	37
3.10	Other features	38
3.11	Charging	38
4. S <sub>l</sub>	pecifications	39
4.1	Dimensions	39
4 2	Sensor	40

## 1. Introduction

Advanced Pressure Mapping System (APMS) is a wireless pressure measuring device. Optimized for measuring the pressure of glass cleaning equipment, APMS helps to keep the equipment in its best condition. The sensor is very thin and completely waterproof, which allows it to measure the pressure on top of the glass. It also uses exclusive calibration device to provide precise and reliable data.


# 2. Package contents

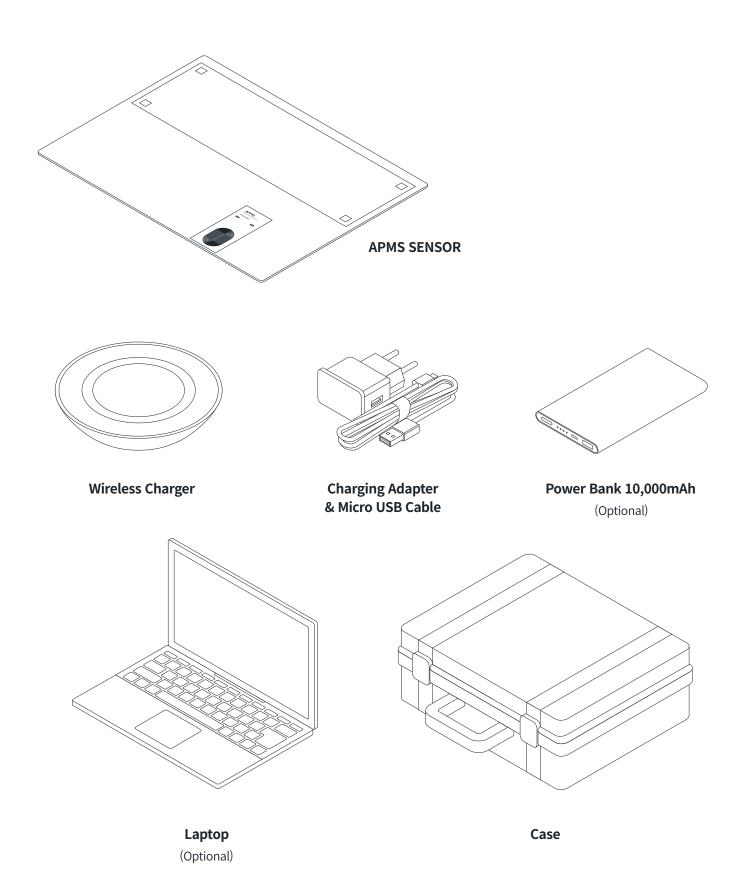
2. Package Contents APMS User Manual


## Warning

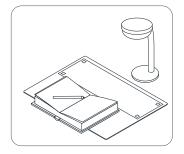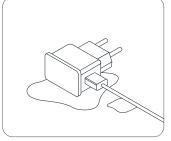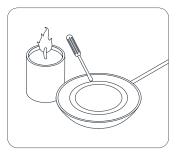- 1. Do not use the device for any purpose other than described in this manual.
- 2. Do not let persons who can not understand this manual to use the device.
- 3. To avoid the risk of electric shock, do not expose the charging adapter or cable to water, liquid or any wet conditions.
- 4. Do not excessively bend, damage, alter, pull, tie, press or twist the charging cable.
- 5. Do not place heavy objects on the charging cable.
- 6. Do not place the charging cable near heat sources or sharp corners of table.
- 7. Use the general power socket for AC adaptor. Using the wrong socket can result in damage to the device.
- 8. Remove dust from the plug and the socket regularly.
- 9. Do not use damaged charging adapter, cable or power socket.





#### Caution

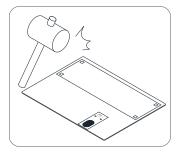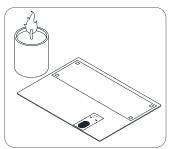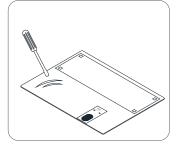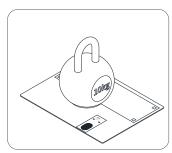- 1. Do not knock over the device.
- 2. Do not drop, throw or kick the device. An impact may cause damage to the device.
- 3. When carrying the device, stop the operation and hold it with both hands to prevent it from bending.
- 4. Do not place the device near pointed or sharp object. It may tear the device and cause malfunction of waterproof.
- 5. Do not step on the device nor place heavy objects on it.
- 6. Do not place flammable object, turn on cigarette lights nor burn incense near the device.
- 7. Do not let children and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge to use this device. They must have given supervision or instruction for the operation by a person who is well-experienced and responsible fort their safety.
- 8. Persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge can use this device only if they have given supervision and instruction to use it correctly and safely and understand the risks and hazards involved.



# 3. Using APMS

## 3.1 Connecting APMS

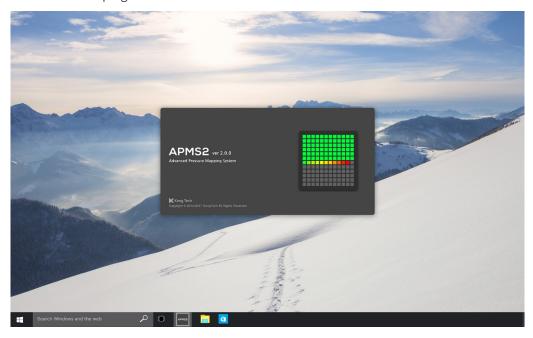
1. Run APMS.exe program.

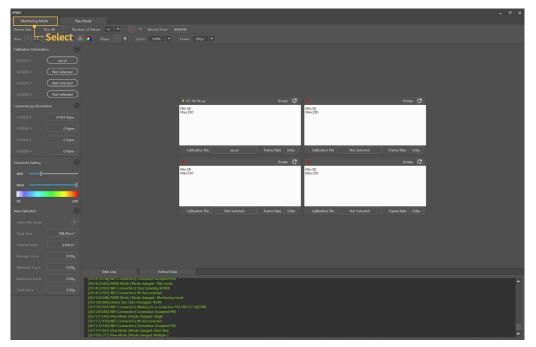
2. Select 'Monitoring Mode' at the top. (Automatically selected when running the program).

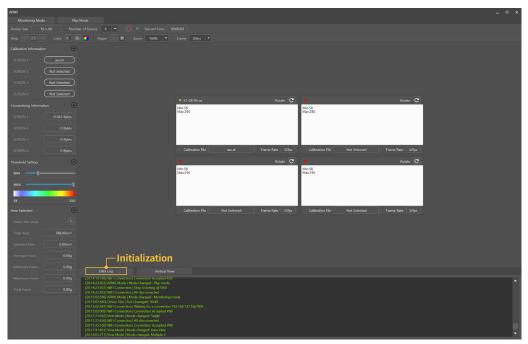- 3. Check the 'Data Log' tab at the bottom to see if the program has completed initialization. (The initialization will be completed when all of the items below are displayed.)
- Wifi Connection I Hotspot started
- Device Size I Size Changed 16 x 48
- Wifi Connection I Waiting for a connection
- View Mode I Mode Changed: Multiple2
- **%** If initialization doesn't work, please contact our customer service (+82 2-1522-9928).

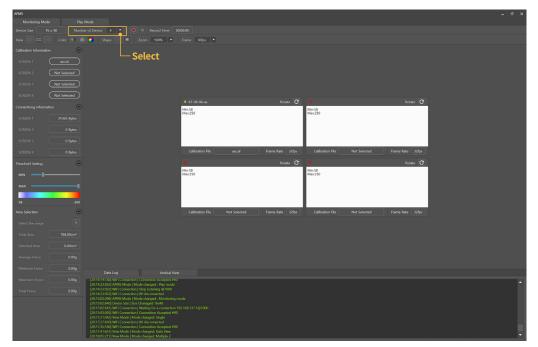
4. Select the 'Number of Devices' to set the number of sensors to use.

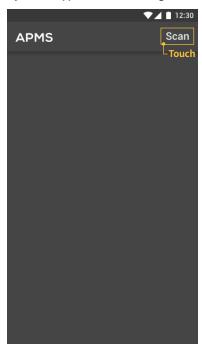- 5. After launching the APMS application, touch the 'Scan' button at the top.
- $\ensuremath{\mathrm{\%}}$  Please install the 'APMS' application from the Playstore or Appstore before starting.


6. Touch the 'Connect' button of scanned sensor. (Information of scanned sensors are displayed.)

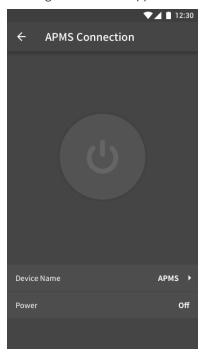
7. Wait until the sensor is connected. (The loading screen will disappear when connection is completed.)

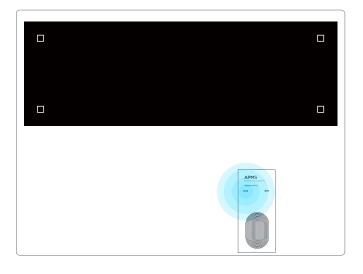
8. Touch the button in the center to activate and the connected APMS sensor will be turned on. (LED light on)

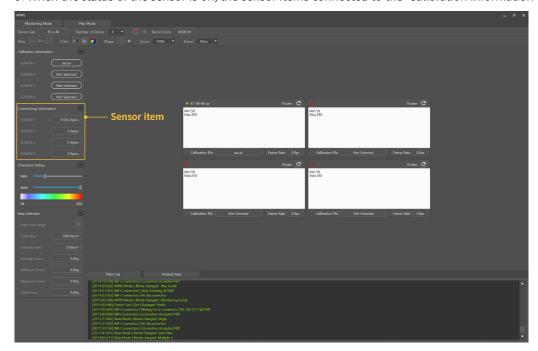9. When the status of the sensor is on, the sensor items connected to the 'Calibration Information' will be automatically set.


10. Once the connection is completed, place the APMS sensor on the glass cleaning equipment to start measuring pressure.


## 3.2 Saving measurement data

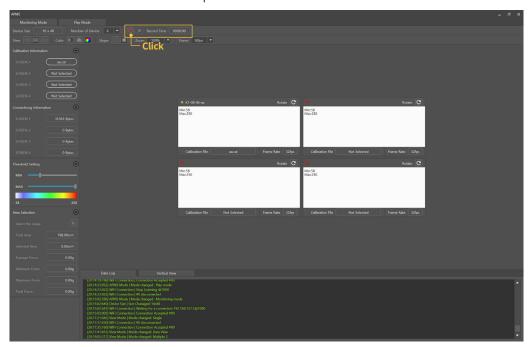The measured data can be recorded and stored.

1. Click the 'Record' button at the top.

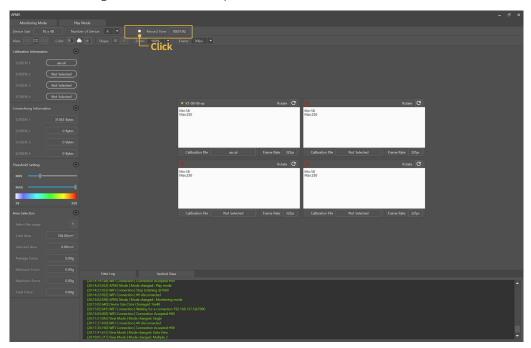2. When recording is done, click the 'Stop' button.


3. Recorded files will be saved in the path below.


Documents/APMS2/Record

## 3.3 Playing recorded files

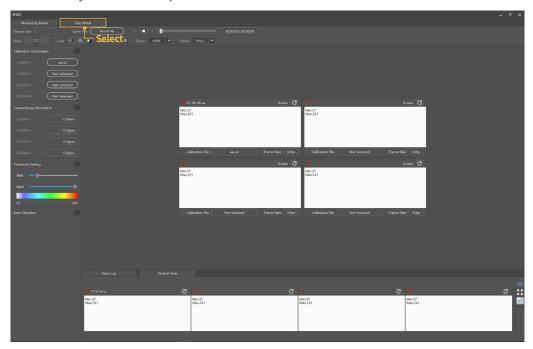1. Select 'Play Mode' at the top.

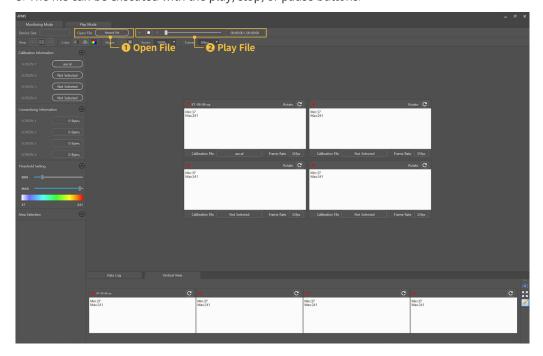
- 2. Select 'Record File' from the 'Open File' tab to select a file to open.
- 3. The file can be executed with the play, stop, or pause buttons.

## 3.4 Changing measurement data viewing mode

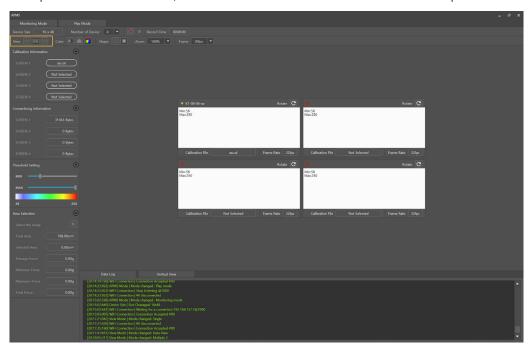1. The measurement data viewing mode has three modes: 'Single View', 'Multiple View' and 'Data View', which can be selected from the 'View' tab at the top.


2. The 'Single View' mode displays one APMS sensor on the screen.

3. The 'Multiple View' mode displays multiple APMS sensors at once on the same screen.

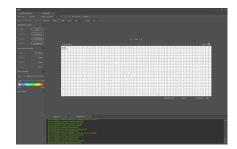
4. The 'Data View' mode displays pressure data received from the APMS sensor on the screen.


## 3.5 Changing the color of measurement data

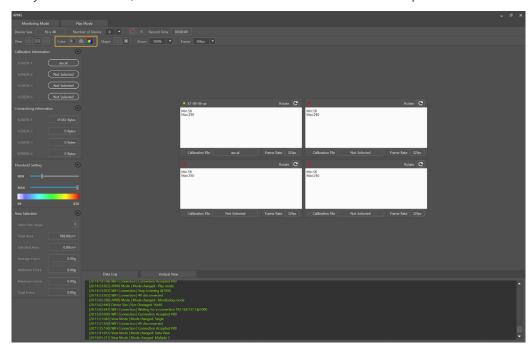
1. Changing the color of measurement data mode has three modes: 'Monotone', 'Grayscale' and 'Color', which can be selected from the 'Color' tab at the top.

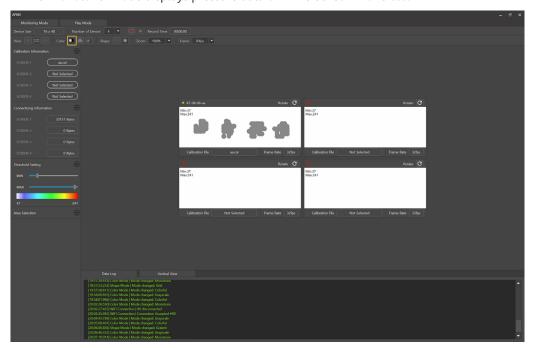
2. The 'Monotone' mode displays pressure data of APMS sensor in one color.

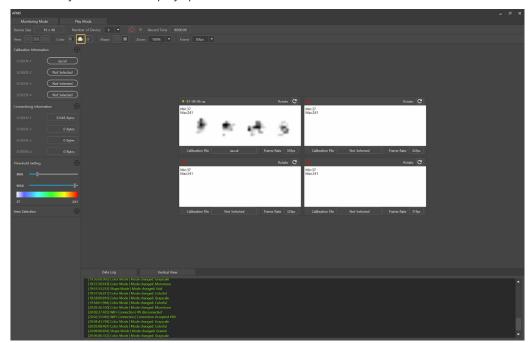3. The 'Grayscale' mode displays pressure data of APMS sensor in black and white.

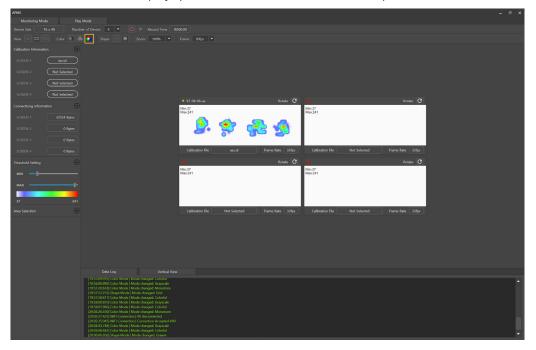
4. The 'Colorful' mode displays pressure data of APMS sensor in multiple colors.


## 3.6 Changing the appearance of measurement data

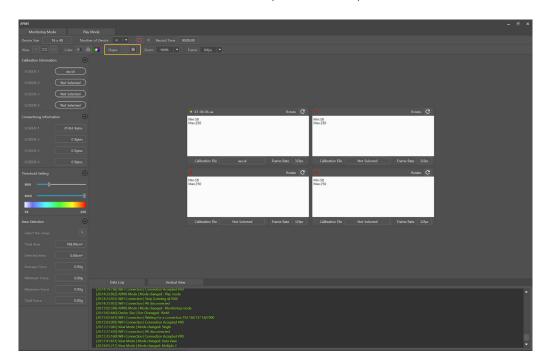
1. Changing the appearance of measurement data mode has two modes: 'Grid' and 'Gradient', which can be selected from the 'Shape' tab at the top.

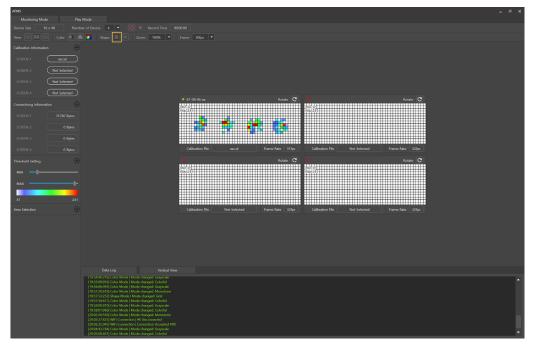2. The 'Grid' mode displays pressure data of APMS sensor in a grid format.


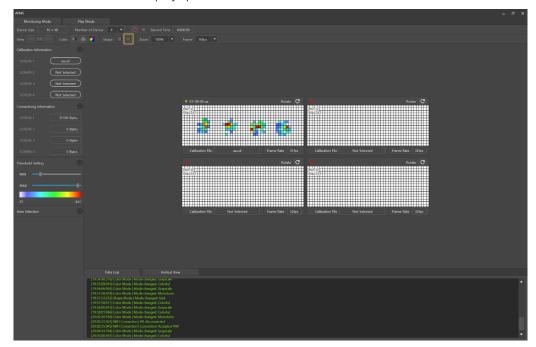
3. The 'Gradient' mode displays pressure data of APMS sensor in a calibrated format.

## 3.7 Changing the measurement range of data

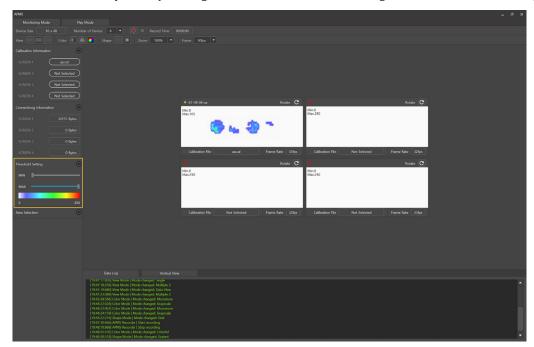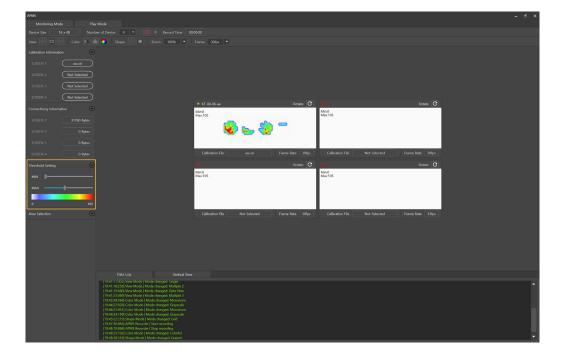
- 1. The measurement range of data can be changed from the 'Threshold Setting' tab on the left.
- 2. Values can be adjusted by moving the bar in the middle or entering a number. The initial setting is MIN: 0, MAX: 100.

## 3.8 Viewing data in a selected range

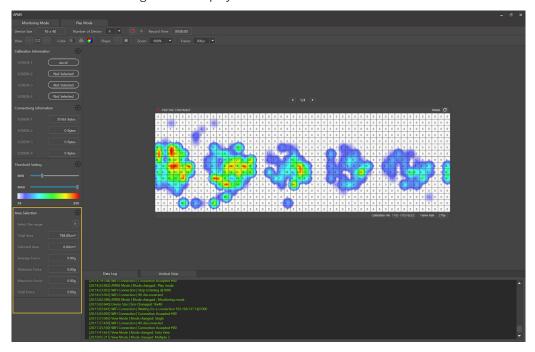
- 1. The data in a selected range can be viewed in the 'Area Selection' tab on the left.
- 2. Click the 'Select the range' button to set the range.
- 3. Data in a selected range will be displayed.

### 3.9 Continuous viewing


Four APMS sensors being measured can be displayed in a row.

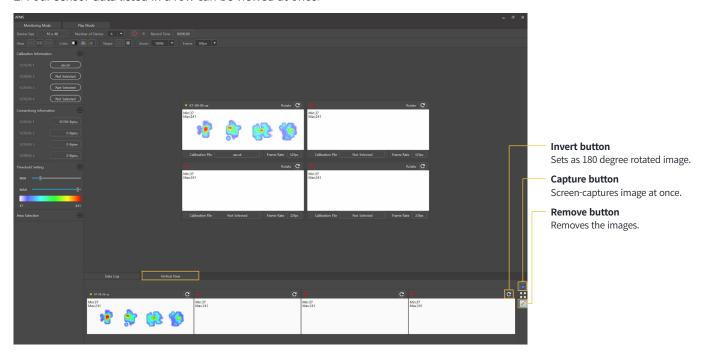- 1. Select the 'Vertical View' tab at the bottom.
- 2. Four sensor data listed in a row can be viewed at once.


#### 3.10 Other features

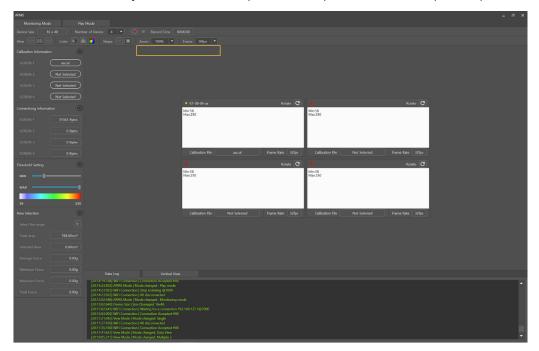- 1. Select 'Zoom' to adjust the screen size from 25% to 150%.
- 2. Select 'Frame' to adjust the frame rate of pressure data per second from 10 fps to 60fps.

## 3.11 Charging

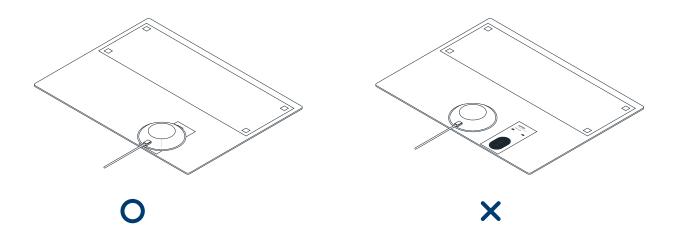- 1. Place the wireless charging pad in the center of the charging area. Make sure it touches the center.
- 2. The status indicator (CHARGER LED) will be turned on when it's in charging status.


## 4. Specifications

#### 4.1 Whole device

#### **Specifications**

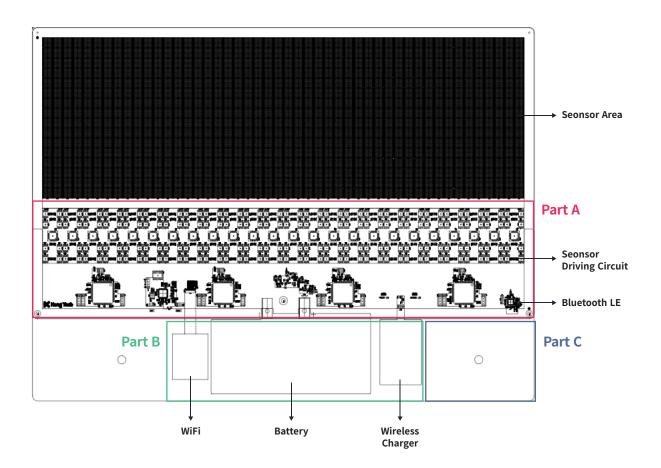Dimensions: W500 X D372 X H1.6(mm) Measuring range: 0~250gf Charging method: Wireless charging Hours of battery use: 3 hours Rated output: 5V / 700mA Current consumption: <800mA Stand by current: <1mA Battery capacity: 2850mAh

# Outer Skin Tape Chip or Carbon Plate PCB Tape Outer Skin

- 1. Outer Skin 110μm(nominal) (Film 90μm and adhesive 20μm)
- 2. Tape 0.6mm(nominal)
- 3. Carbon plate 1mm

Part A

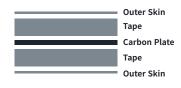
- 4. Chip(max 1.6mm)
- 5. PCB 0.6±0.15mm

# Outer Skin Tape WiFi, Wireless Charging Pad, Battery(max height) Carbon Plate Tape

Outer Skin

- 1. Outer Skin 110μm(nominal) (Film 90μm and adhesive 20μm)
- 2. Tape 0.6mm(nominal)
- 3. Battery 1.7mm
- 4. Carbon plate 0.6mm


#### Part C


- 1. Outer Skin 110μm(nominal) (Film 90μm and adhesive 20μm)
- 2. Tape 0.6mm(nominal)
- 3. Carbon plate 1.6mm


4. Specifications APMS User Manual

#### 4.2 Sensor

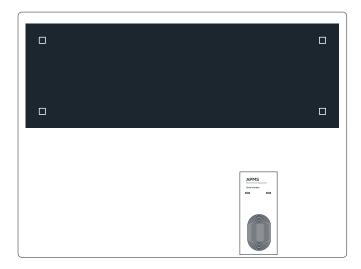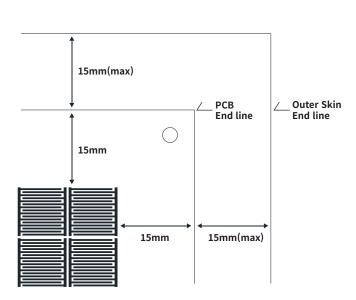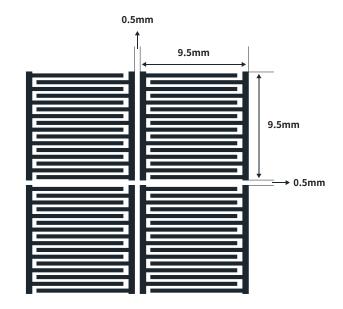- 1. The area covered with black film is the part that measures the pressure.
- 2. There are 768 built-in pressure sensors.
- 3. The white squares in the corners are zero-point reference points for calibrating.

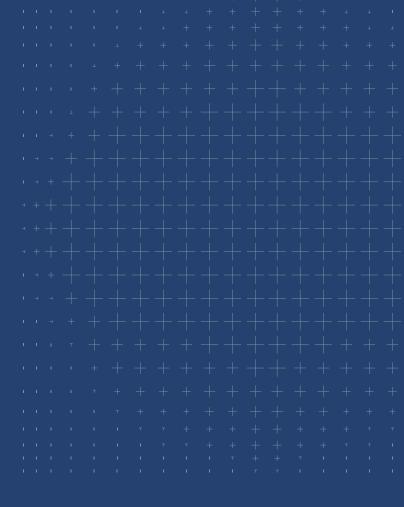
- 4. The spacing between the sensor, the PCB and the edge is 15mm.  $\,$
- 5. The size of sensor patterns is 9.5 x 9.5 mm.
- 6. The spacing between patterns is 0.5mm.

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